

# CLEAR FLEX

The milled PREMIOtemp  
**CLEAR FLEX** bite splint from primotec

## Care instructions:

- The PREMIOtemp CLEAR FLEX splint should be cleaned under running water with a toothbrush after each use
- Store your bite splint in a dry and dark place if not worn
- Do not use denture cleaners / alcohol based cleaners
- Place the bite splint in warm water at 40-50 °C for 1-2 minutes before insertion
- Protect the splint from extreme heat and coldness



**primotec**<sup>®</sup>

Joachim Mosch e.K. Tannenwaldallee 4 D-61348 Bad Homburg  
Tel. +49 (0) 61 72 - 99 77 0 - 0 Fax +49 (0) 61 72 - 99 77 0 - 99  
primotec@primogroup.de www.primogroup.de