

# THE BITE SPLINT WITH THERMO-EFFECT



**excited  
patients**

**pleased  
dentists**

**relaxed  
technicians**

## primoflex CLEAR

**Nothing presses, nothing straints**

**the new primoflex CLEAR bite splint is just  
super comfortable to wear.**

## PATIENT INFORMATION



# primoflex CLEAR



# MAXIMUM BREAKAGE RESISTANCE WITH THERMO-EFFECT

PRIMO



**Maximum breakage resistance** at body temperature through thermoplastic flexibility

## TOP FEATURES

## YOUR BENEFITS



**Memory-Effect:** 100% reset at body temperature without loss of surface hardness

- Tension-free wearing comfort and precise fitting due to the thermo-effect
- Very durable – maximum breakage resistance through memory effect – more resistant to breakage than most conventional PMMA plastics
- High transparency for almost invisible wearing
- Very low plaque affinity

# primoflex CLEAR

# primoflex CLEAR

**The new primoflex CLEAR  
bite splint from primotec**

## **Care instructions:**

- The primoflex CLEAR splint should be cleaned with toothpaste after each use
- Store your bite splint in a glass of water if not worn
- Do not use denture cleaners
- Place the bite splint in warm water at 40-50 °C for 1-2 minutes before insertion
- Protect the splint from extreme heat and coldness



**primotec**<sup>®</sup>  
DIGITAL

Joachim Mosch e.K.  
Tannenwaldallee 4  
D-61348 Bad Homburg  
Tel. +49 (0) 61 72 - 99 77 0 - 0  
Fax +49 (0) 61 72 - 99 77 0 - 99  
[primotec@primogroup.de](mailto:primotec@primogroup.de)  
[www.primogroup.de](http://www.primogroup.de)